


Electric Lift Chair Assembly Instruction_ 9008#

Find a small accessories box inside the iron frame. Take out it.
Assemble the spare parts to the chair.





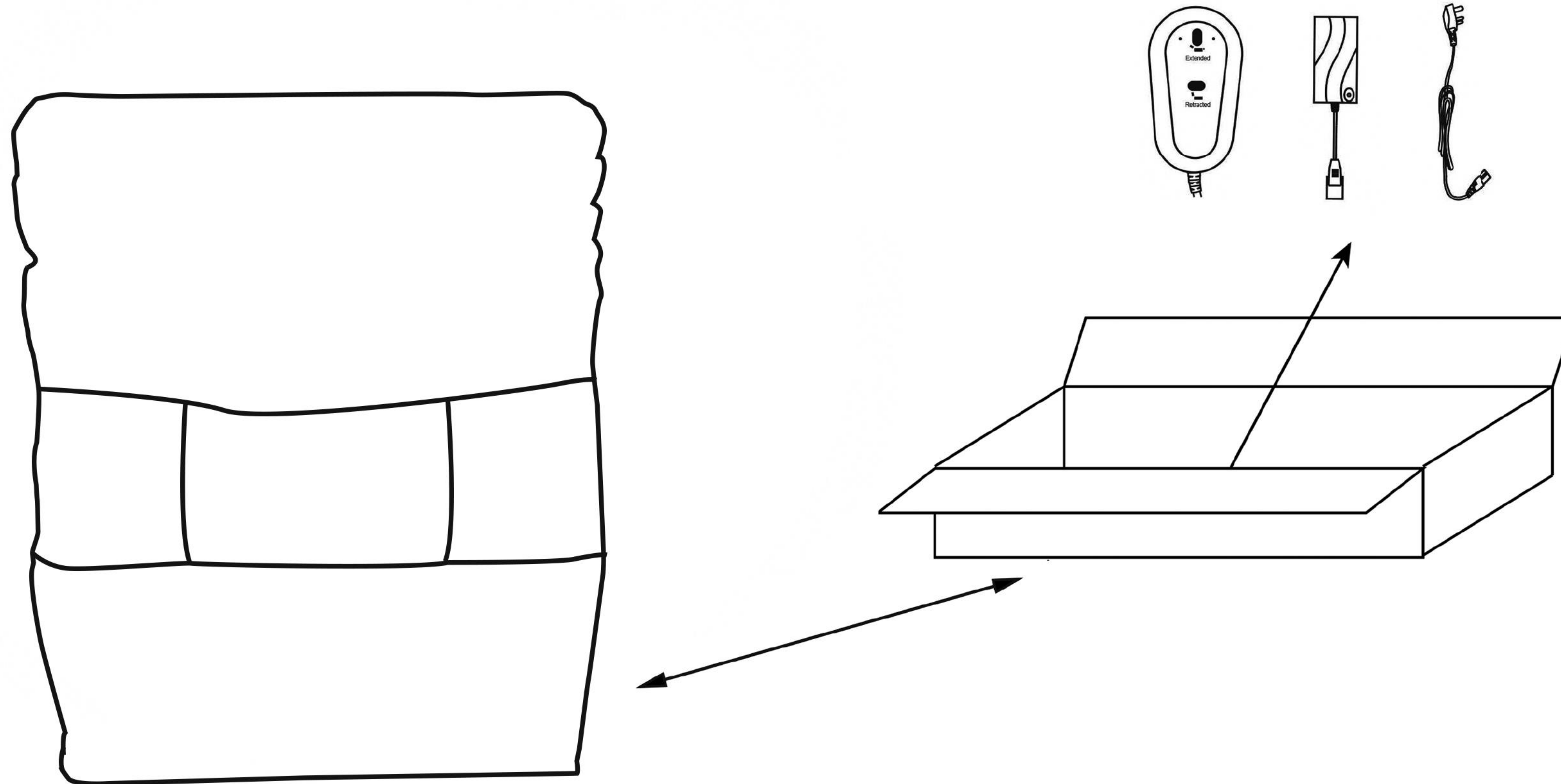





You will receive 2 boxes, please make sure you get one (A) carton and one (B) carton before assembling.


All boxes may not arrive on the same day, it usually takes a few days, so please be patient and then assemble after you have received all the packages. Assembly will be done after all packages are received.

If you have any questions, please feel free to chat with us and we will try our best to help you solve the problem.

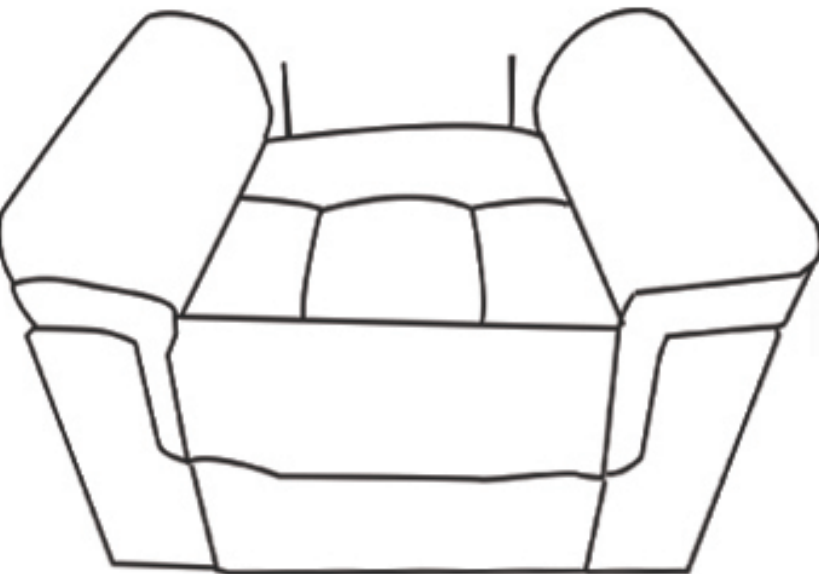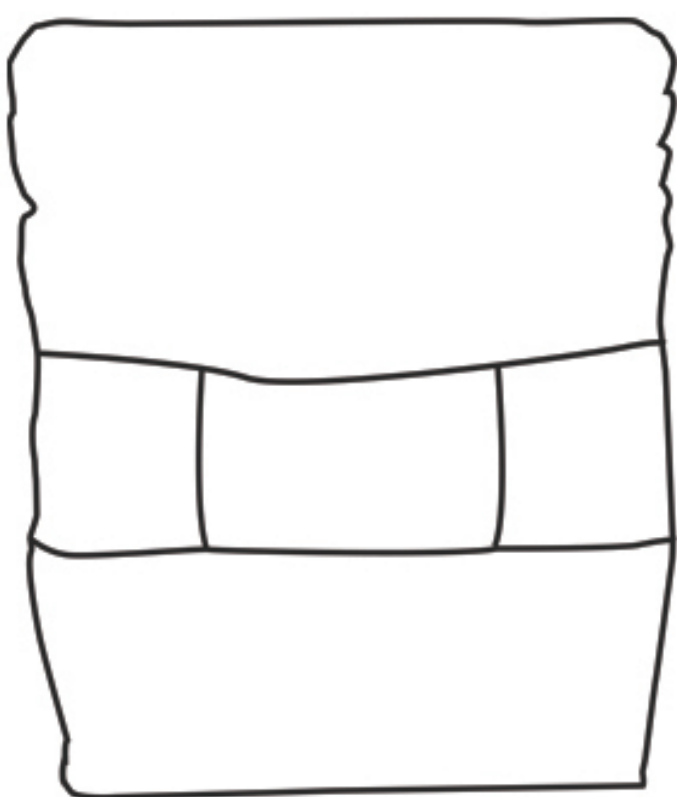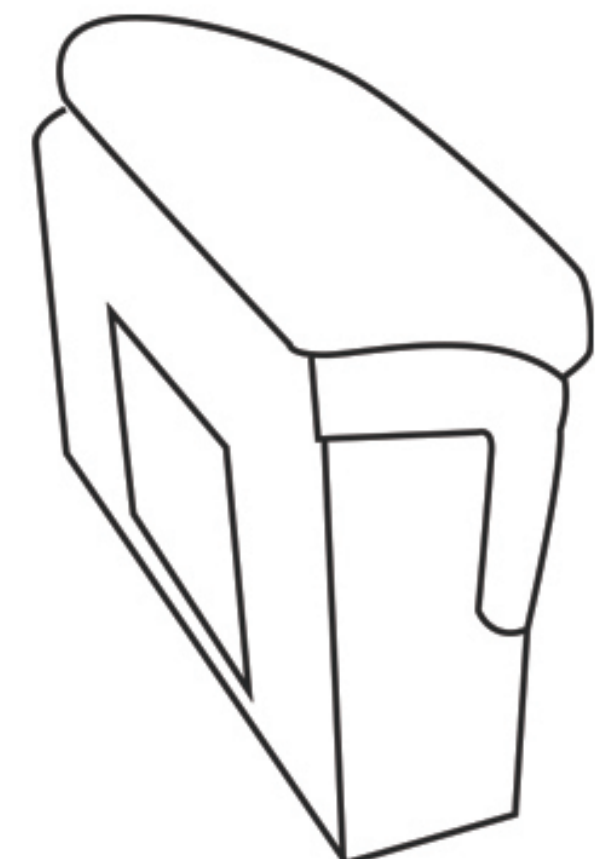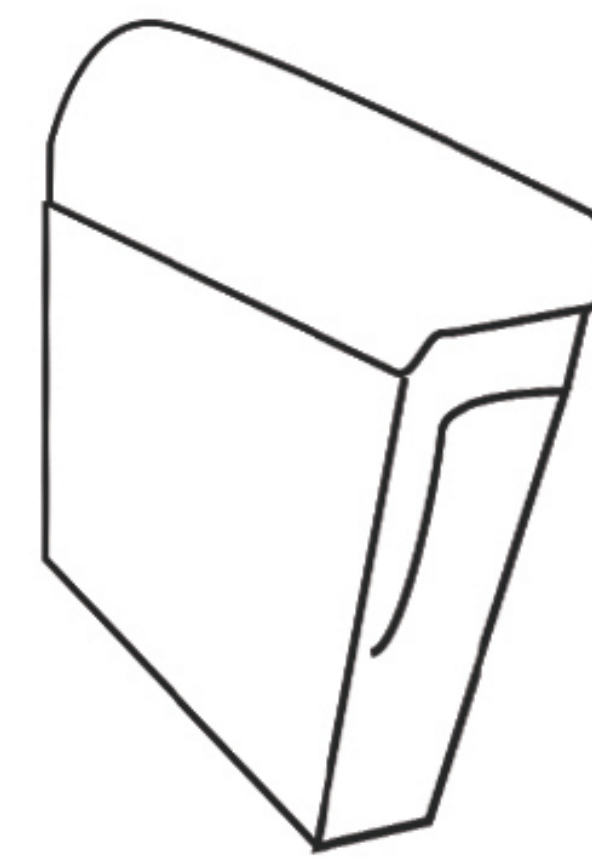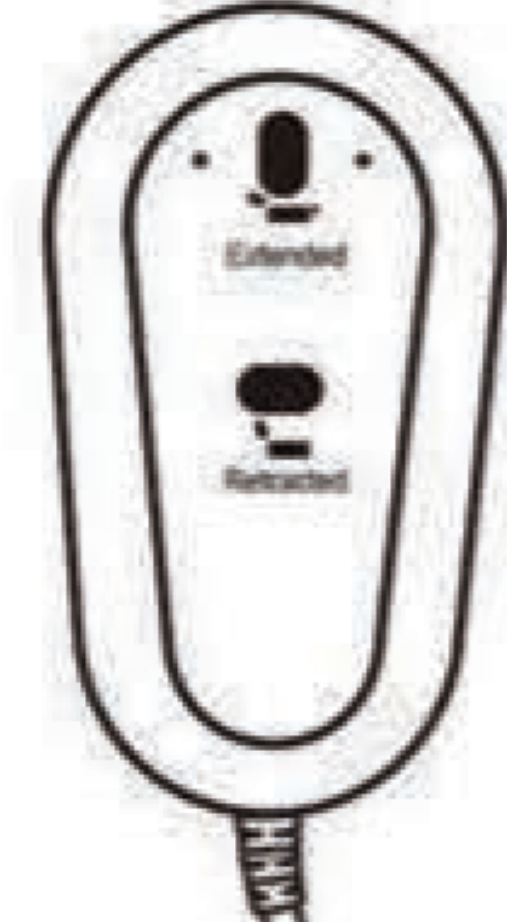
Electric Lift Chair Assembly Instruction_9008#

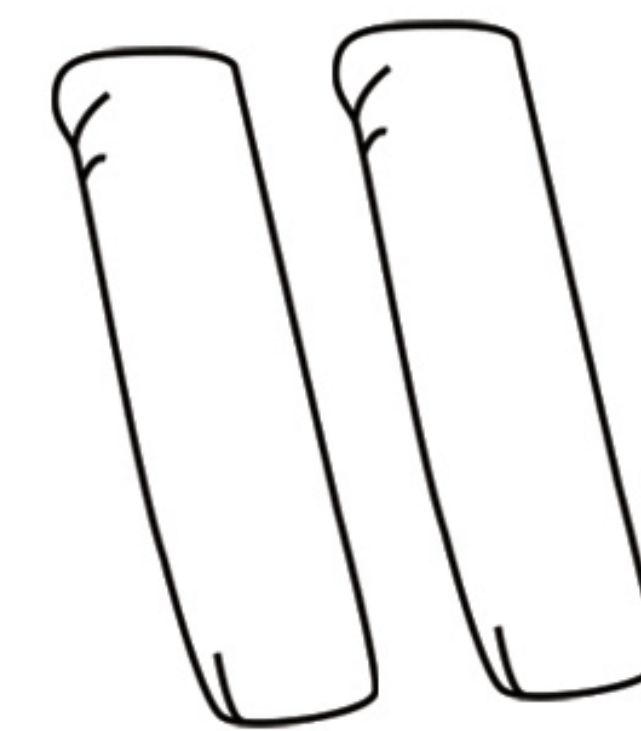								
Chair base	Backrest	Left Armrest	Right Armrest	Chair Remote	Chair Remote	Power Source	Left and right wing	


STEP 1

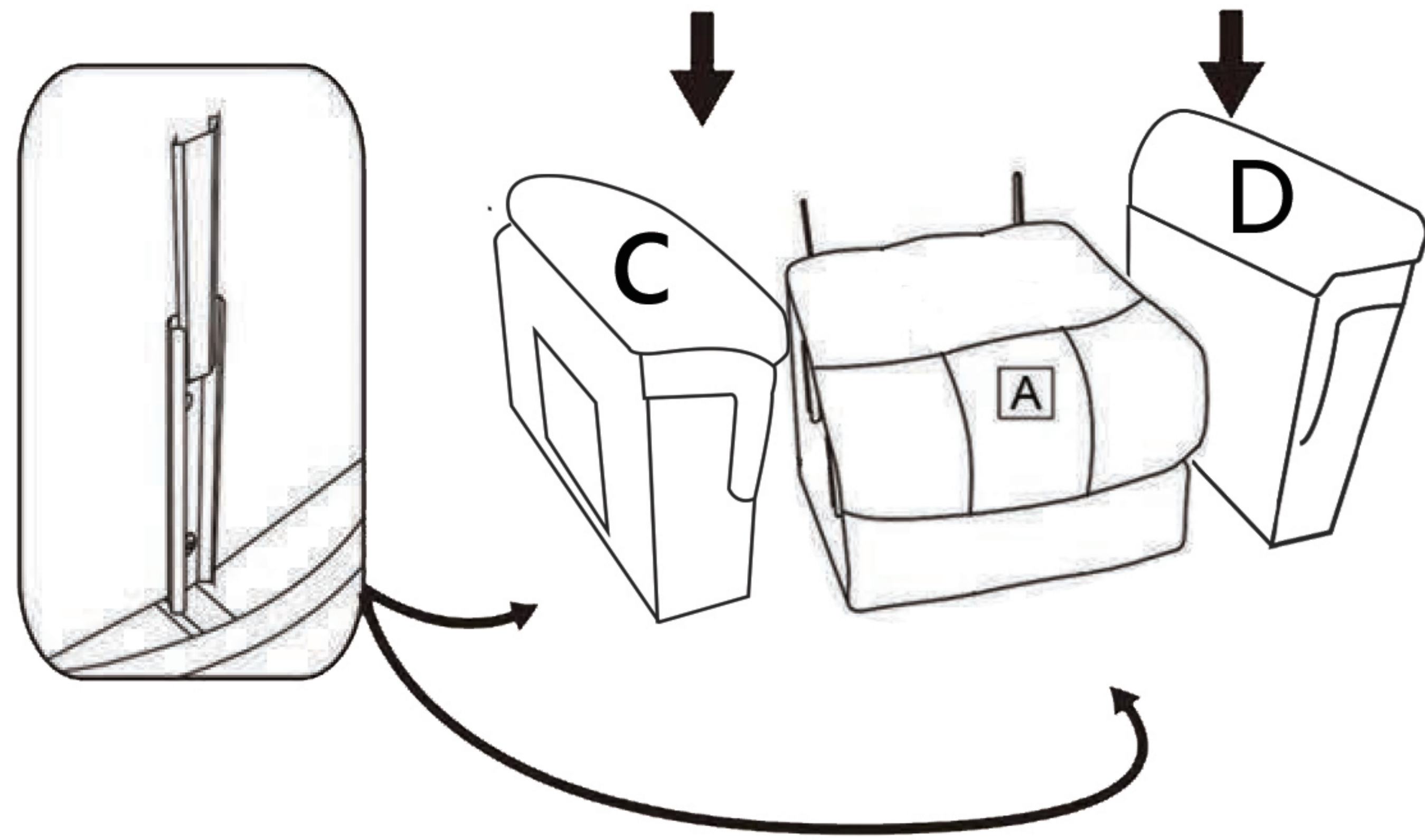


Insert two armrests to the chair base, make sure the slots on the armrests fully assembled to the bottom part.

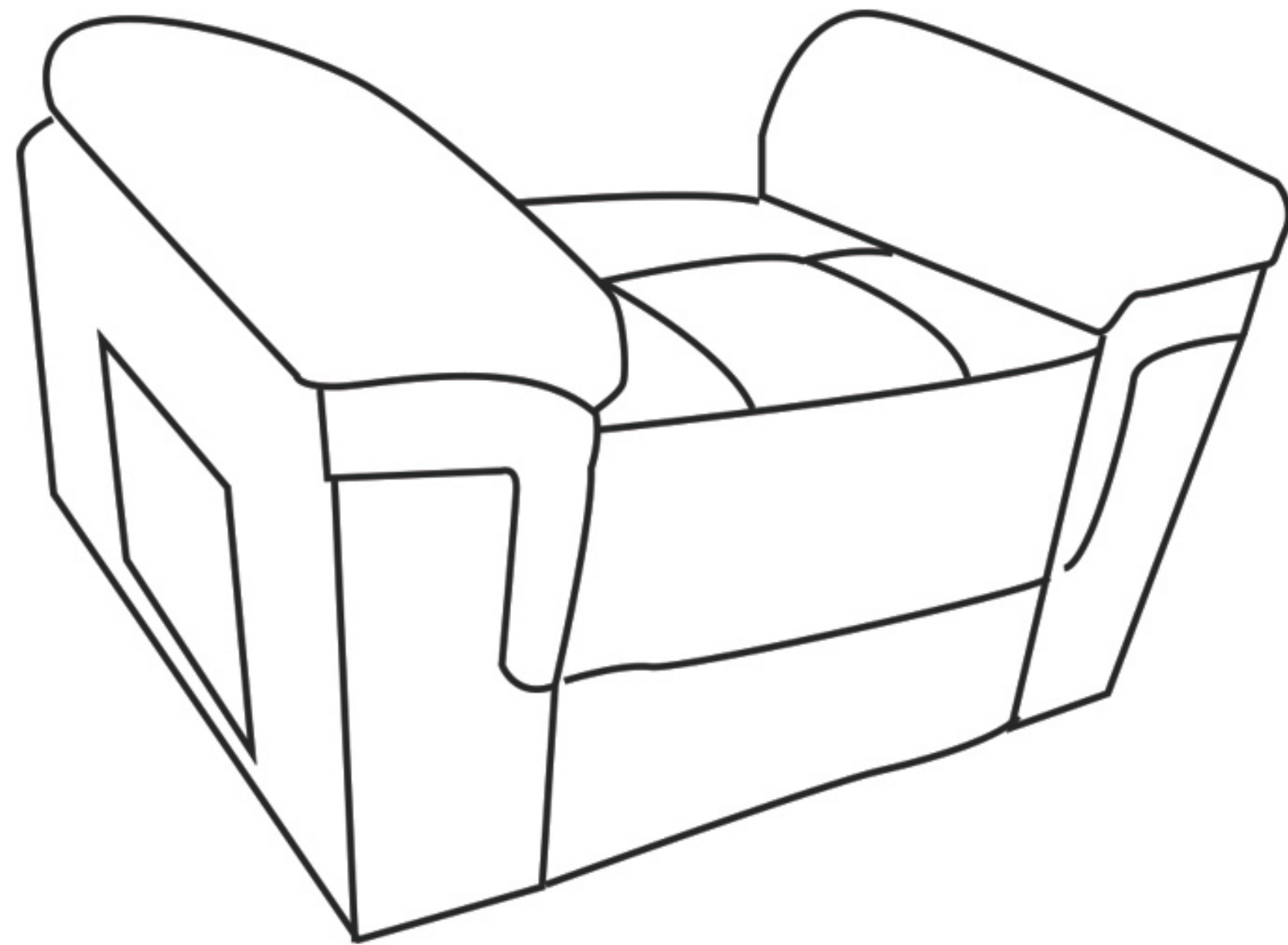

STEP 2

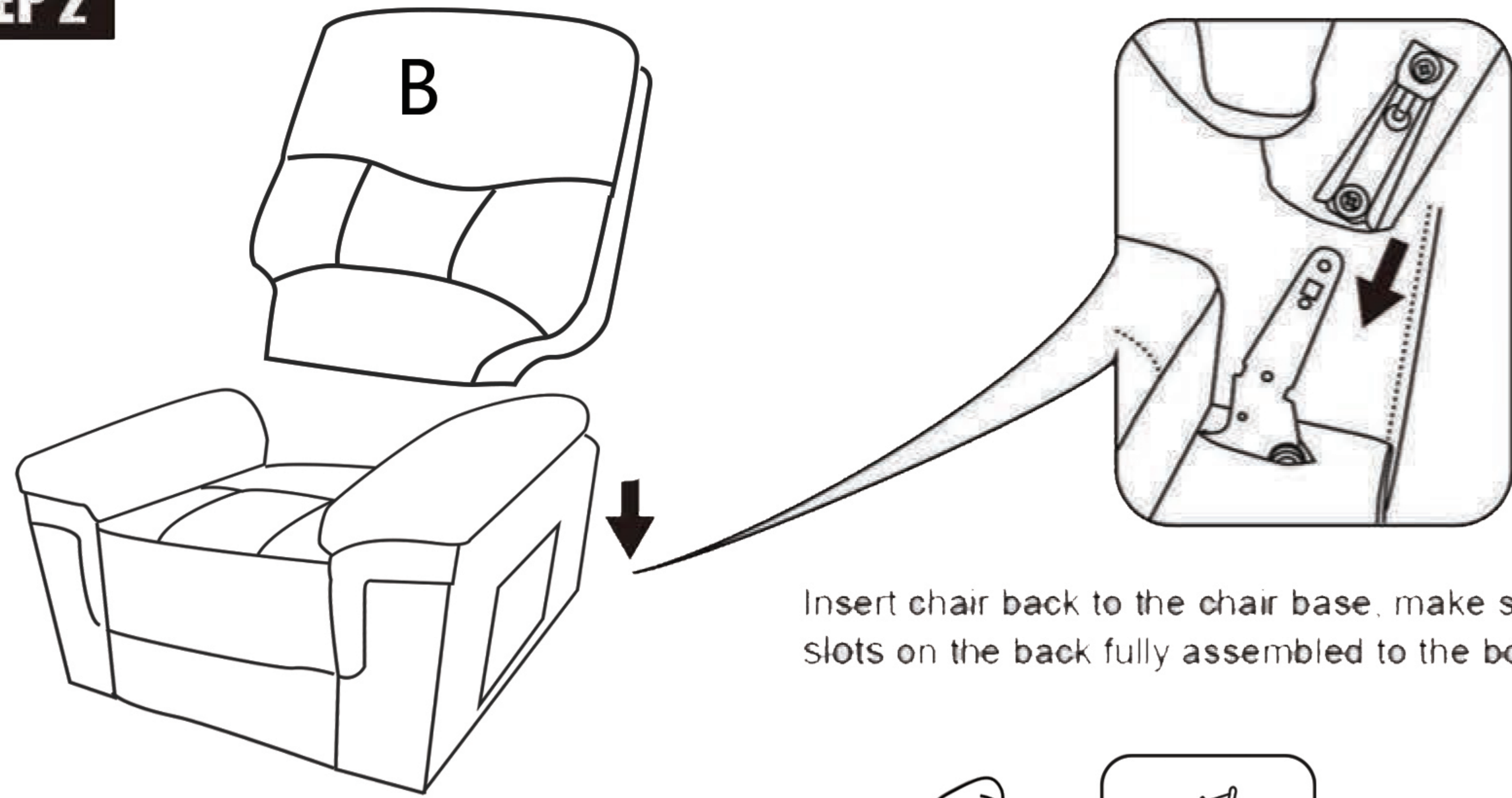

Insert chair back to the chair base, make sure the slots on the back fully assembled to the bottom part.

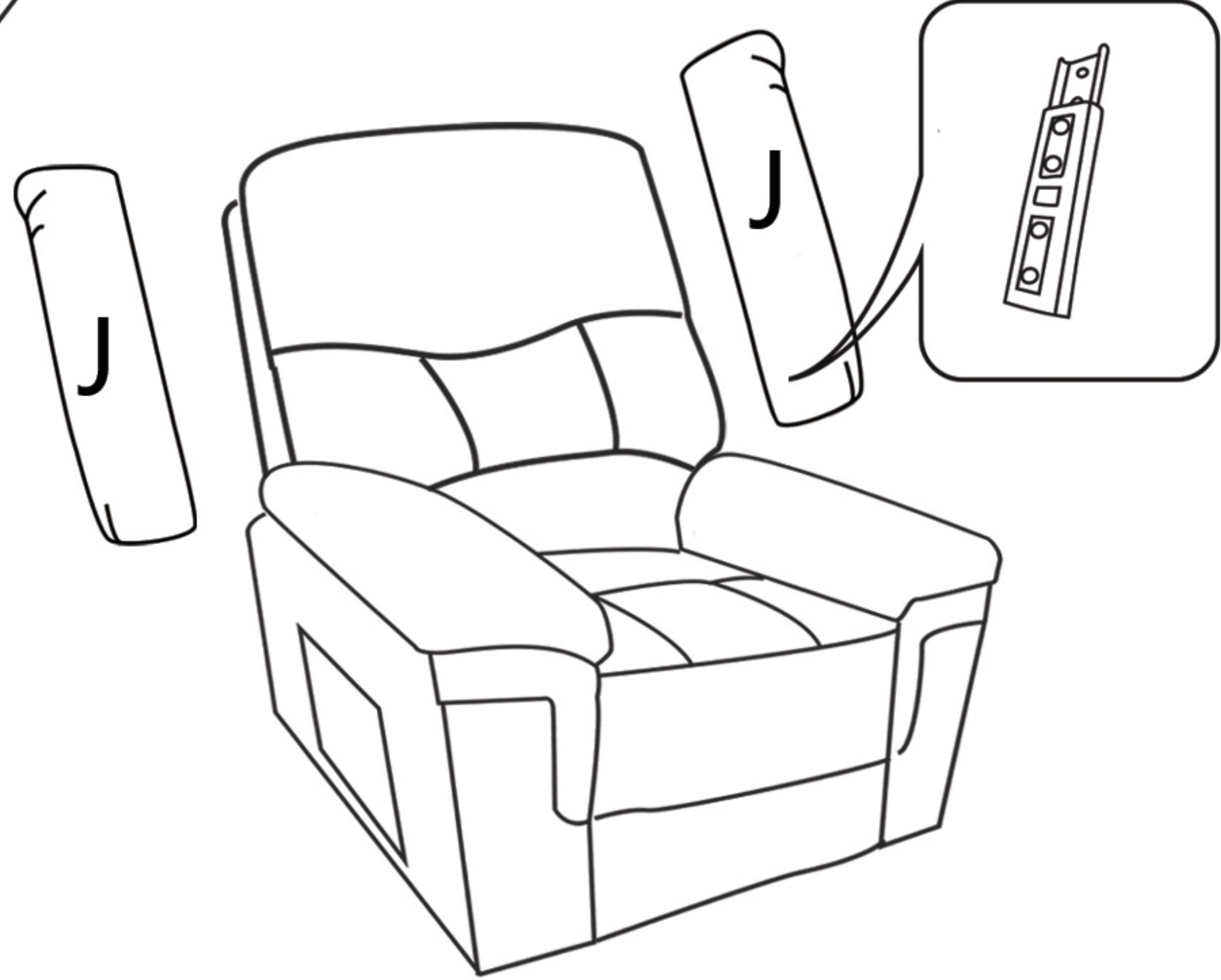

**UK
CA**

Electric Lift Chair Assembly Instruction_ 9008

STEP 3


No.1 wire connect the motor and the wire from Chair Remote(G), which use to connect the control panel.

No.2 wire connect the motor and power source (H).

NOTE


1: Two same color wires connect.

2: Plug power cord and power source into outlet, then you can use all function.

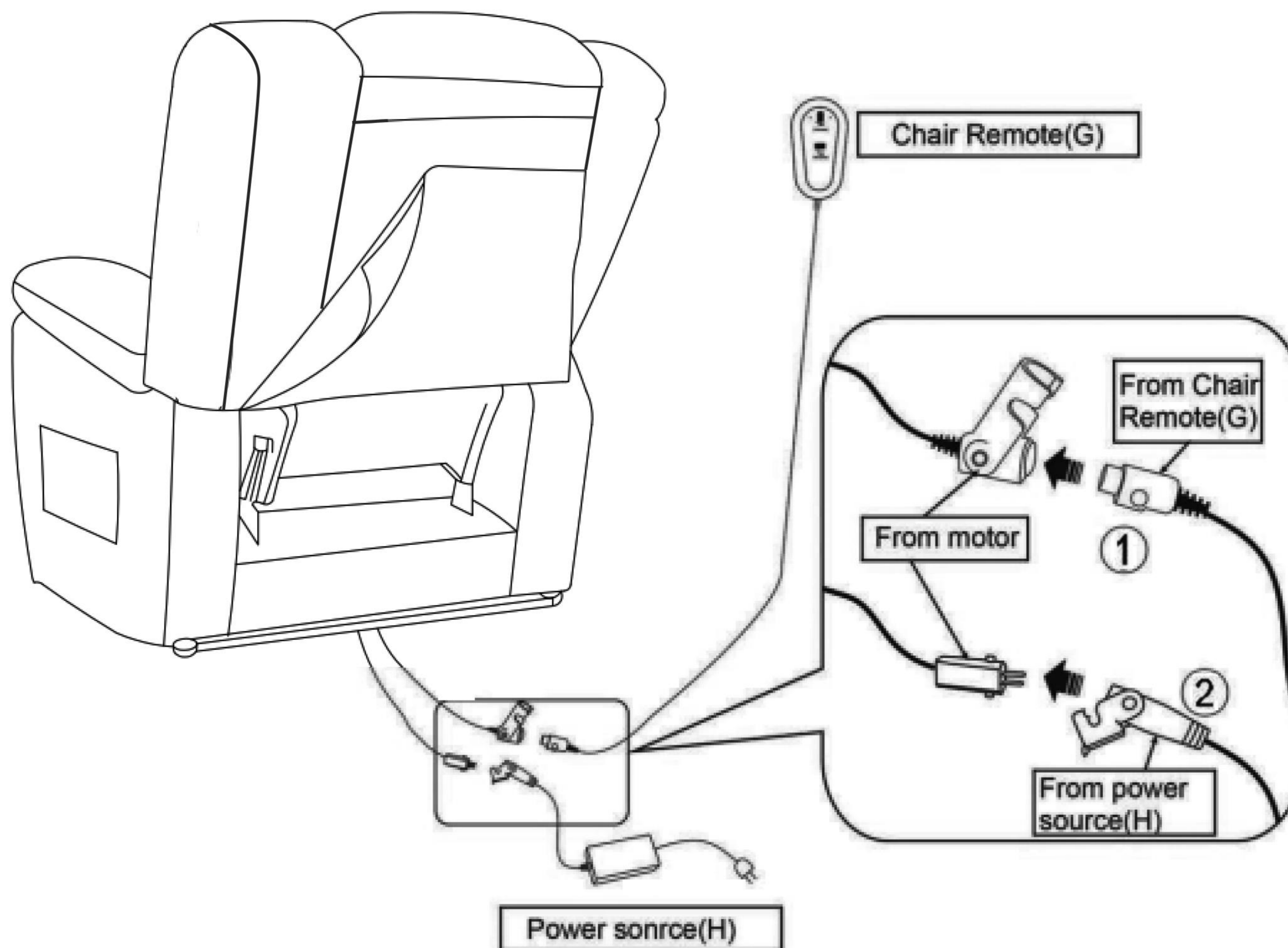

Electric Lift Chair Assembly Instruction_9008#

How to recline?

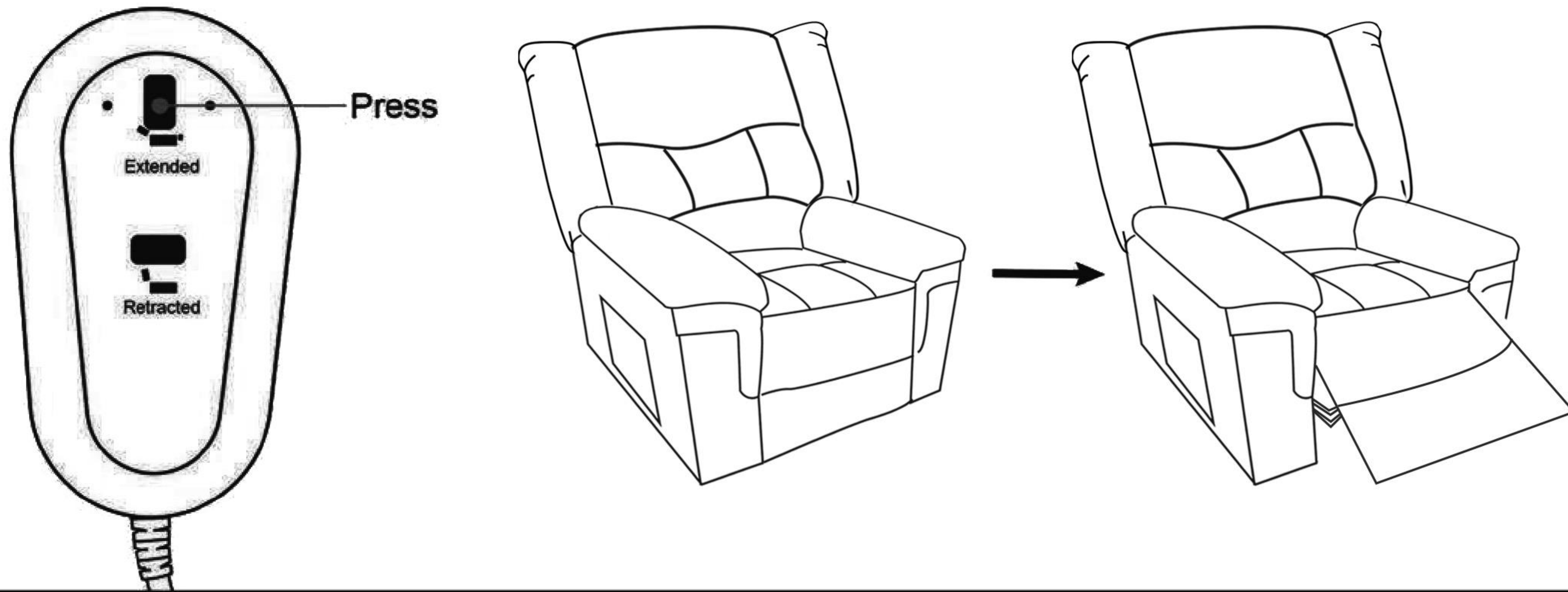



IMPORTANT!!!



Cables Crush Hazard!

Before using the chair, please extend the cable under and out behind the chair, tidy up the connected wires to avoid wrapping, causing pinching and breakage

How to get footrest latch down and stand?

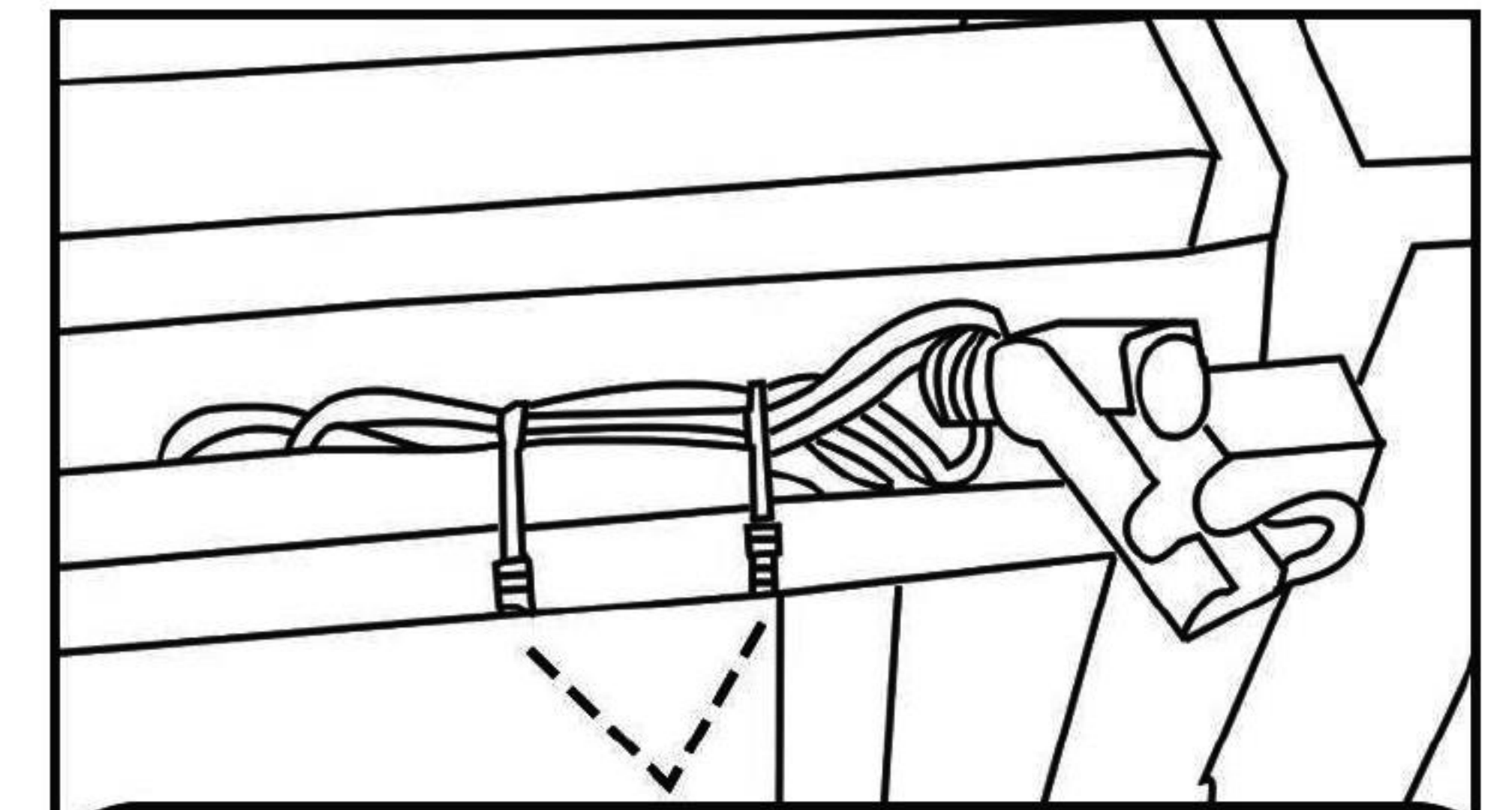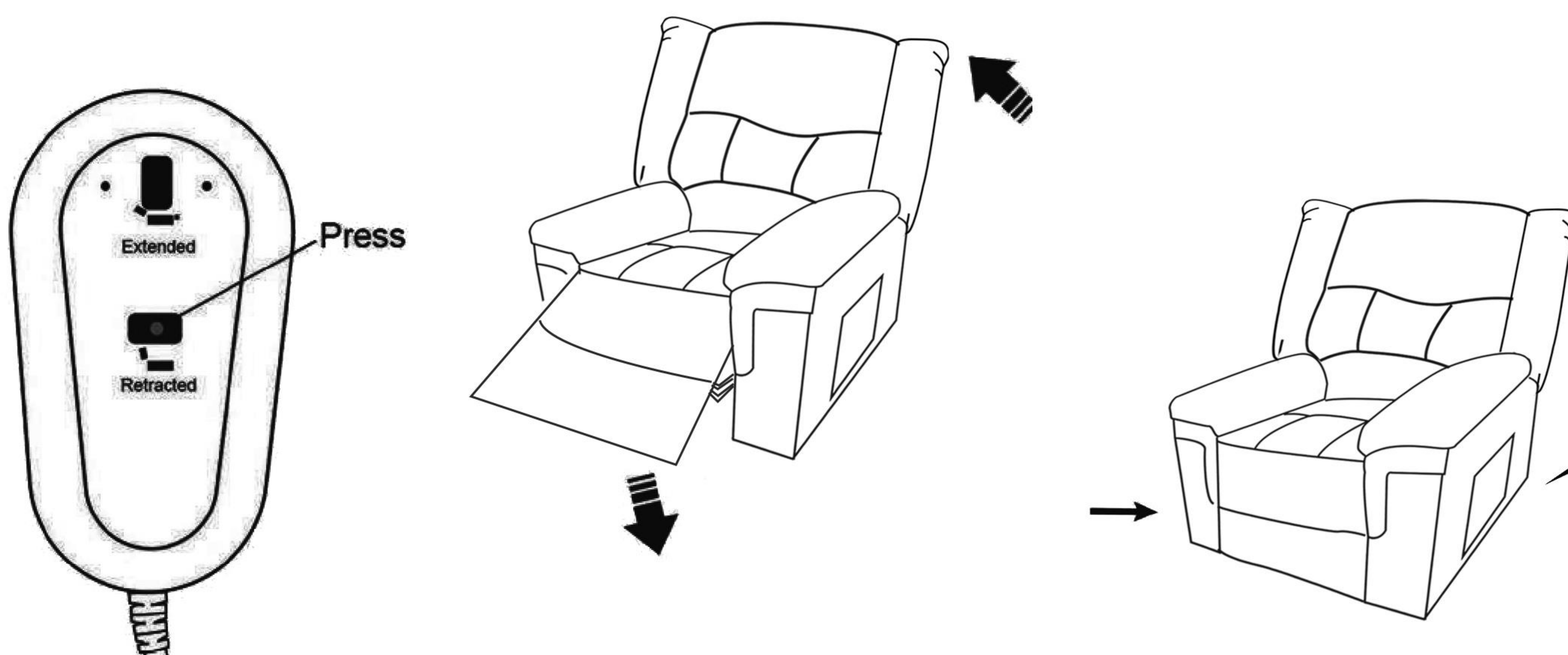

Before using the chair, please tidy up the connected wires to avoid winding the base and causing pinch off

**UK
CA**