

WhizzerTM



STEP BY STEP
INSTRUCTIONS



1

Once you open the box your whizzer will look like this. Everything you need is inside. Peel the lid off (no screwing needed)



NB: The lid must be OFF to expand or collapse due to suction

2

A) Remove the white stability ring with the detachable funnel

B) You'll find the unisex spout beneath



3

The raised bumps on the spout fit into the holes in the stability ring. Ensure they're lined up and secure before use (it gets quicker with practice)

Ensure the highest part is at the front as a privacy shield.



Loop for carry strap



Unisex spout

scoops down to body.

New Leakproof funnel

design with widened hole for fast fit beneath the flow fillers is in new whizzers. All parts are available separately.

4



Once used, either remove the spout, stability ring and funnel all in one to empty immediately. Revert steps back to state 1. Or, pinch the spout off and replace with the lid and put both parts in a bag to empty/clean later.



- * Clean with hot soapy water after use.
- * Dishwasher safe.
- * Use sterilising tablets (e.g. Milton) for deep clean.
- * Use nothing abrasive.
- * Bags and straps sold separately